PIPETTE / SERINGUE DOSEUSE

DOS'UP®, DOS'UP® PLUS

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DOS'UP®, DOS'UP® PLUS
DOSING PIPETTE/SYRINGE
Paediatric syrup and oral solution dosing
INSTRUCTIONS FOR USE

DESCRIPTION

Please read these instructions before use.

DOS'UP® and DOS'UP® PLUS Dosing Pipettes/Syringes are CE-marked Medical Devices, indicated for dosing and dispensing paediatric syrups and oral solutions. They are composed of a barrel and plunger assembly, printed with dosing graduations on one of its components.

Dosing Pipettes/Syringes are manufactured by UNION PLASTIC by injection moulding of European and U.S. Pharmacopoeia-compliant food contact plastics. Some references may be dyed using a food colour.

Dosing Pipettes/Syringes are only used with the medicinal product with which they are packaged and according to the dosage specified by the manufacturer in the package leaflet. Dosing Pipettes/Syringes are reusable, by and for the same patient, throughout the treatment with the medicinal product.

USE

Before the first use, it is strongly recommended to *rinse* the Dosing Pipette/Syringe with *cold water*. *The Dosing Pipette/Syringe must not be disassembled during rinsing.*

There are two possible modes of use: with or without adaptor.

1. A) With adaptor

• Check before use:

- Check that the Dosing Pipette/Syringe is printed, the graduations are legible, the adaptor is fitted in the Dosing Pipette/Syringe and none of the components of the Dosing Pipette/Syringe are damaged,
- Open the bottle,
- Insert the adaptor and therefore the Dosing Pipette/Syringe into the neck of the bottle until it comes to a stop,

• Dosing:

- Turn the bottle upside down,
- When dosing, it is recommended to hold the bottle and the Dosing Pipette/Syringe firmly,
- Pull the pipette/syringe plunger slowly to draw up the medicinal product, to the line corresponding to the dosage indicated in the package leaflet, or in the healthcare professional's prescription.
- Return the bottle to the upright position and remove the Dosing Pipette/Syringe from the adaptor. The adaptor remains in the bottle.

NB: The adaptor is an accessory of the Dosing Pipette/Syringe used to facilitate dosing.

B) Without adaptor

• Check before use & Dosing:

- Check that the Dosing Pipette/Syringe is printed, the graduations are legible, and none of the components are damaged,
- Open the bottle,
- Immerse the pipette/syringe in the medicinal product, checking that the plunger is pressed down fully,
- Pull the plunger slowly to draw up the medicinal product, to the line corresponding to the dosage indicated in the package leaflet, or in the healthcare professional's prescription.

2. Dispensing / Use:

- Aim the pipette/syringe inside the cheek and release the contents of the dosing pipette/syringe into the child's mouth by pushing slowly and regularly on the plunger.

3. After dispensing / Use:

- *Leave the Dosing Pipette/Syringe assembled. Do not* separate the Dosing Pipette/Syringe barrel and plunger.

4. Cleaning

After each use:

- *Rinse the Dosing Pipette/Syringe with cold water, pumping it several times until* any residual medicinal product has been removed.
- Allow the *Dosing Pipette/Syringe* to air-dry,

5. Storage

- Store the Pipette/Dosing Syringe with the bottle of medicinal product in the original box, out of reach of children.

INDICATIONS

Dosing Pipettes/Syringes must be used according to the procedure described by the manufacturer in the package leaflet.

Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

CONTRAINDICATIONS

- Do not use the Dosing Pipette/Syringe if it has not been cleaned properly or is damaged.
- Do not use the Dosing Pipette/Syringe for any other application or for any other medicinal product than the one included or prescribed.
- Do not use to dispense in other orifices.
- Do not use the Dosing Pipette/Syringe for any other patient.
- *Do not wash the Dosing Pipette/Syringe in the dishwasher.*
- *Do not boil the Dosing Pipette/Syringe.*
- *Do not put the Dosing Pipette/Syringe in the microwave.*
- The Dosing Pipette/Syringe may not be used after the medicinal product expiration date.

DEGREE OF DOSING ACCURACY OF THE MEDICAL DEVICE

The dosing accuracy is 5% for Dosing Pipettes/Syringes with a total dosing capacity of ≤1mL.

The dosing accuracy is 10% for Dosing Pipettes/Syringes with a total dosing capacity of > 1mL.

PRECAUTIONS FOR USE

DO NOT USE THE DEVICE IF IT IS DAMAGED, NOT PRINTED OR IF THE GRADUATIONS ARE NOT CLEARLY LEGIBLE.

The Dosing Pipette/Syringe must not be used by anyone of unsound mind or with cognitive impairment; anyone using the device must be capable of reading and understanding these instructions.

Keep out *the Dosing Pipette/Syringe* of reach of children.

SPECIAL STORAGE CONDITIONS

The Dosing Pipette/Syringe must be kept clean and dry in its original packaging at a temperature between 5°C and 50°C, away from direct sunlight.

The shelf life of the Dosing Pipette/Syringe is 5 years.

The Dosing Pipette/Syringe must be disposed of with household waste.

MANUFACTURER INFORMATION

First marketing approval: 1998

Last revision date of instructions for use: Instructions DT3 revision F dated 2026/02/25

CE Marking identification Notified Body: GMED 0459